

## CASTRGARD



The best way to move anaesthesia machines and wheeled OR carts safely without wheel obstructions.

- Durable one-piece design with integral living hinge, locking tab, closure positioning/support pins.
- Easy to install and remove without a tool.
- Protects expensive cables, probes & wires, leading to significant cost savings
- Eliminates the need to stop monitoring your patient to clear a wheel obstruction.
- Excellent chemical and impact resistance.



### General Use

The closed and locked position is the proper operating position. Place CASTrGARD on all four wheels. To transport clinical machines across uneven surfaces (such as carpet) or gaps in the floor (e.g. elevators or expansion plates), all devices must be removed to avoid tipping over machines while moving.



### Removal (Opening)

To open and remove the device, find and turn the locking latch toward the body. Place the right thumb on locking tab and push in. With the left hand, hold the left half of device and pull out and away from the right half. The device may now be removed from around the wheel. Repeat for each wheel.



### Installation (Closing)

To apply the device, place CASTrGARD in open position around the wheel. Bring both halves of the device together until locking tab locks into the opposite side. Locking tab must be fully seated in locking tab opening with edges of device flush together. Verify the device is placed on the floor with the UP arrow pointing upward and the correct label orientation.




### Storage (Closed)

The closed and locked position is the correct way to operate devices, as well as to store devices not in use or upon removing the devices from machines prior to transport.

### TECHNICAL SPECIFICATION

<b>A45SM</b>	<b>Caster Size:</b> 12.7 cm diameter maximum with 7.9 cm minimum height clearance <b>Overall Size:</b> 18.0 cm outside diameter x 7.6 cm high; 14.5 cm inside diameter <b>Weight:</b> 0.68 kg each
<b>A456MD-4B</b>	<b>Caster Size:</b> 15.2 cm diameter maximum with 11.7 cm minimum height clearance <b>Overall Size:</b> 19.1 cm outside diameter x 11.4 cm high; 16.3 cm inside diameter <b>Weight:</b> 0.73 kg each
<b>A67LG</b>	<b>Caster Size:</b> 18.1 cm diameter maximum recommended 12.7 -> 18.1cm with 9.3 cm minimum height clearance <b>Overall Size:</b> 21.8 cm outside diameter x 9.1 cm high; 18.2 cm inside diameter <b>Weight:</b> 0.86 kg each

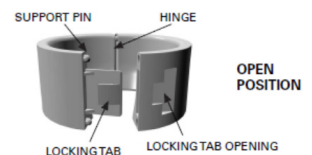

Product Number	Description	QTY
A45SM-4B	Cast-R-Gard Wheel Guard, Blue, Small, Caster Size to 12.7cm	Set 4
A456MD-4B	Cast-R-Gard Wheel Guard, Blue, Medium, Caster Size to 15.2cm	Set 4
A67LG-4B	Cast-R-Gard Wheel Guard, Blue, Large, Caster Size 12.7->18.1cm	Set 4