



## Check CLIP

### A simply safer way of labelling syringes

Checkclip safely attaches the ampoule to the syringe for accurate cross check without obscuring incremental dosage marks on the syringe to reduce medication errors. Checkclip enables the ampoule to travel with the syringe for visual cross-checking and accurate administration to the patient.

- Enables visual cross checking of drug and dose details
- Ensures a clear unobscured view of syringe increments
- Prevents sharps injuries from opened glass ampoules
- Provides clear indication of drug expiry dates



### CheckCLIP FOR AMPOULE

1. Face barb down and push Checkclip on to syringe using your thumb.

2. Snap open the ampoule and draw up the prescribed medication. Without the ampoule or syringe leaving your hands, proceed to step 3.

3. Push ampoule up on to Checkclip.

4. Double check before injecting.

### CheckCLIP FOR VIAL

1. Face barb down and push Checkclip on to syringe using your thumb.

2. Draw up the prescribed medication and without the vial or syringe leaving your hands, proceed to step 3.

3. Using the needle track as a guide, push vial up on to Checkclip.

4. Double check before injecting.

Product Code	Description	QTY
QSCCGEN	Check Clip for Syringe & Ampoule	Box 100
QSCCVIAL	Check Clip for Vial	Box 100