



Endoscopy Masks & Accessories

Allows simultaneous ventilation and endoscopy of the patient to increase patient comfort and the working quality of



Fiberoptic Intubation:

The ET Tube can be railroaded orally or nasally along the bronchoscope through the flexible membrane. After intubation the membrane is pushed inside which allows easy removal of the mask

Bronchoscopy:

The membrane guarantees an ideal seal and therefore an efficient ventilation before, during and after bronchoscopy without time pressure.


Gastroenterology

Trans Esophageal Echocardiography:

The use of Entonox (50% N2O, 50% O2) is ideal for minor procedures where pain is predictable and of short duration. Entonox can be self administered from the conscious patient via the mask. It has rapid analgesic effect and therefore allows a pain free insertion of the Endoscope through the hole in the membrane. Entonox wears off quickly without post operative problems.



Description	QTY	Product Code
Endoscopy Mask for single use, Material: PVC Size 0, Newborn 2.0 mm	Each	30-40-000
Endoscopy Mask for single use, Material: PVC Size 1, Infant 2.0 mm	Each	30-40-111
Endoscopy Mask for single use, Material: PVC Size 3, Child 3.0 mm	Each	30-40-333
Endoscopy Mask for single use, Material: PVC Size 3, Child 5.0 mm	Each	30-40-335
Endoscopy Mask for single use, Material: PVC Size 5, Adult 5.0 mm	Each	30-40-555
Endoscopy Mask for single use, Material: PVC Size 5, Adult 10.0 mm	Each	30-40-777

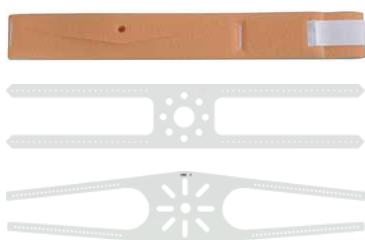

Description	QTY	Product Code
Fixation tape for single use, Material: Isoprene, Adult	Box 10	35-30-255
Fixation tape autoclavable, Material: Silicone, Child	Box 10	35-70-155
Fixation tape autoclavable, Material: Silicone, Adult	Box 10	35-70-255