



Disposable Haemorrhoid Banding Ligator - LEM

Lem4Rings, the last born of Sapi Med products, completes the extensive range of disposable haemorrhoid ligation devices.

Quick and easy to use, Lem4Rings has 4 preloaded rings that allow the performance of multiple elastic ligatures during a single outpatient visit.

Directions:

1. Connect Lem4Rings to a low pressure surgical suction device (5-10 mm H₂O, 50-100 Pa).
2. Place the elastic ligature on the haemorrhoidal nodule to be tied.
3. Gently suction the lump.
4. Slowly rotate the toothed ring to make the first ring drop to the base of the nodule.
5. The procedure can be repeated to perform up to 4 ligatures, if necessary.



LATEX FREE Rings in version A.5680LF

LEM for use with SUCTION

Suction tubing connects the LEM to a low-pressure suction aspirator, and the LEM is then inserted into an operative proctoscope and placed over the surgical site. The haemorrhoid is retracted from the tissue by suction and then banded using the trigger-mechanism. During the procedure, the rate of suction may be controlled manually by the practitioner's thumb. Latex free. Non sterile.

LEM for use with Forceps

In place of suction, grasping forceps are inserted through a cut-out opening in the barrel of the bander and used to manually retract the haemorrhoid for banding. Latex free. Non sterile.

Product Code	Description	QTY
A.5680	LEM - Disposable haemorrhoid ligation suction and banding instrument with 4 latex rings	Each
A.5680LF	LEM - Disposable haemorrhoid ligation suction and banding instrument with 4 latex-free rings	Each