



Camera Drape

The Next Level Of Camera Drapes

High Definition Camera Drape

- Suitable for high-definition camera systems without compromising the HD image quality
- Closed-end offers cameras full protection from fluids and acts as a reliable sterile barrier
- Allows quick and easy scope changes during procedures without compromising sterility
- Broad-edged ring ensures ease of use and prevents contamination of the sterile field
- Sterile, Single use



Step 1: Attach the endoscope. Simply push the endoscope into the drape holder.



Step 2: Attach the camera. Connect camera onto the camera drape holder and extend the drape over the cable by pulling back the ring. Any excess drape is retained inside the ring.

Quick Change Camera Drape

- Simple and secure 2-step fixing method.
- Allows quick and easy scope changes during procedures without compromising sterility
- Closed-end offers cameras full protection from fluids and acts as a reliable sterile barrier
- Broad-edged ring reduces risk of contaminating the sterile field.
- Sterile, Single use



Step 1: Attach the camera. Simple push-fit. The drape can then be extended over camera and cable by simply pulling back the ring. Any excess drape is retained inside the ring.



Step 2: Attach the endoscope. Simple and secure snap-on fitting.

Product Code	Description	QTY
ECE-150-250C	Quick Change Camera Drape - 150mm x 2.5m Ring Aperture 85mm	Box 50
ECE-150-250CNS	Quick Change Camera Drape 150mm x 2.5m Single use Bulk Pack	Box 200
ECN150-250CAP	High Definition Camera Drape, 150mm x 2.5m, Ring aperture 85mm	Box 50