





## Introducer's

Adult Paediatric and Neonate size Teflon rod with rounded tips for use prior to intubation.  
Suitable for tubes down to 2.5mm.I.D

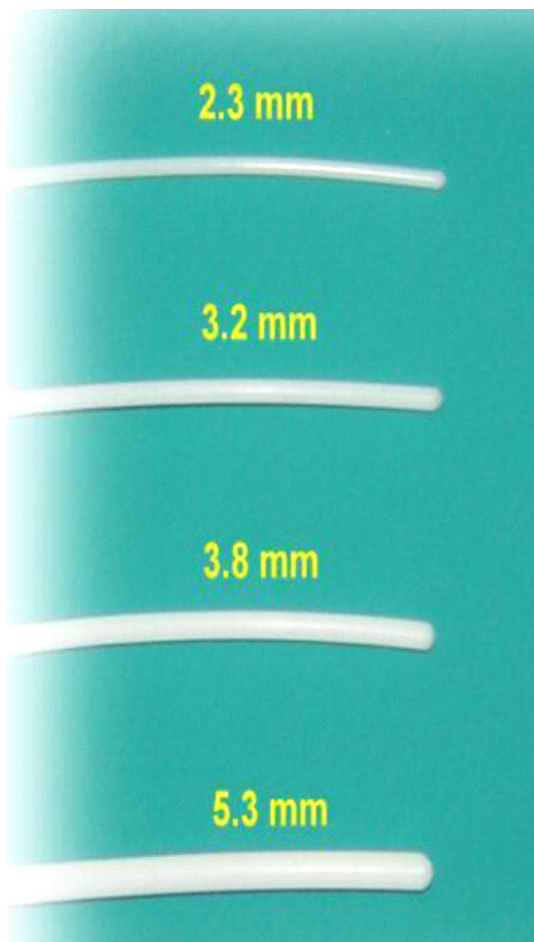

The smooth and “slippery” surface along with the turned edges reduce the risk of trauma and almost eliminate the need to lubricate these products prior to insertion especially when they are generally going into a moist environment. If some lubricant is used even sterile water they become almost frictionless. Some customers use a strip of tape at one end for grip and labelling.

Although this product is malleable with reasonable memory it may not be strong enough to allow shaping of ET Tubes to the extent desired however it does allow the shaped rod to be inserted first through the difficult passage and then the ET Tube can be guided over the rod into position. The malleable feature may deteriorate with repeated autoclaving/heat processing.

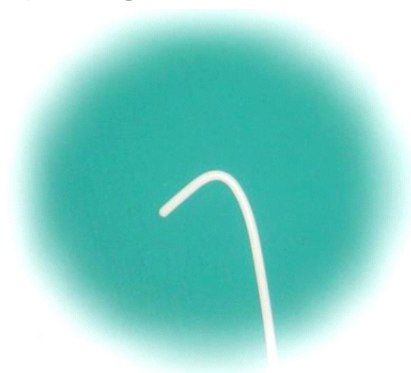

AUSTRALIAN MADE  
PRODUCT OF AUSTRALIA

**LATEX  
FREE**

Product Code	Product Description	QTY
21502	Introducer Neonatal Small 2.3mm by 500mm	Each
21503	Introducer Neonatal Large 3.22mm by 500mm	Each
21504	Introducer Paediatric Short 3.8mm x 500mm	Each
21505	Introducer Adult Short 5.3mm x 500mm	Each
21704	Introducer Paediatric Long 3.8mm x 700mm	Each
21705	Introducer Adult Long 5.3mm x 700mm	Each