




Dilagent

DILAGENT is a soft silicone anal dilator. It is indicated for the treatment of anorectal diseases caused by a hypertonic sphincter, namely anal fissures, haemorrhoids and painful spasms after surgical treatment of the anorectal segment. It is also effectively used in cases of postsurgical stenosis of the anal canal.




Healing treatment of fissures, hemorrhoids, hypertonia and postoperative stenosis

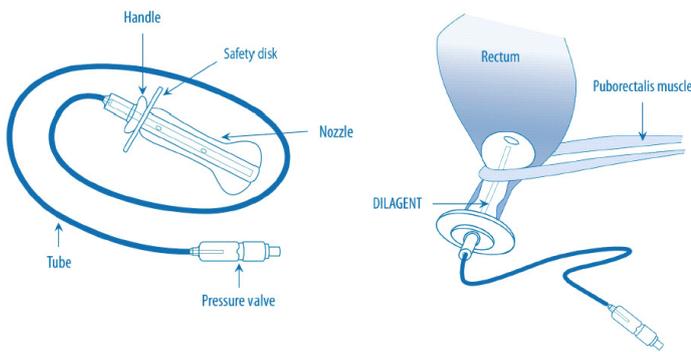

How To Use

1. A hip bath with warm water and Sapigen K for 5-8 minutes is recommended before treatment to relax perianal muscles.
2. After using the provided syringe to remove any air contained in the probe, generously lubricate the dilator with Dilatan Cream and gently insert it into the anus until the handle comes in contact with the anal rim.
3. For the first 2 or 3 days, inject 10-20 mL of hot water into the probe (40°/45°).
4. As treatment proceeds, gradually increase the quantity of warm water to a maximum amount of 50 mL.
5. Larger amounts might cause a continuous bothersome urge to defecate. The length of the tube enables easy performance of said manoeuvres autonomously.

Treatment of vaginismus and dyspareunia.

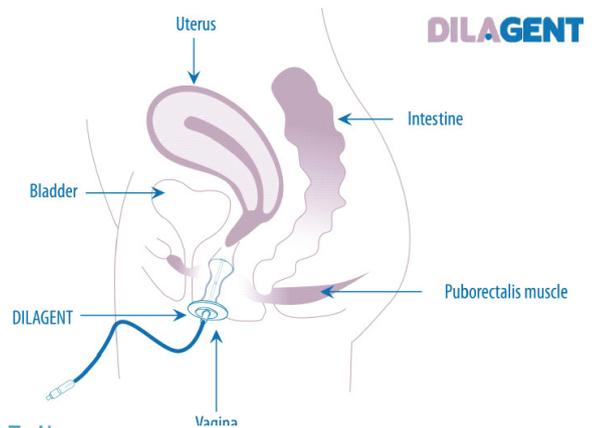

How To Use

DILAGENT has proven to be important in the treatment of vaginismus, which is characterised by pain upon penetration of the penis due to involuntary spasm of pelvic muscles surrounding the distal third of the vagina and to contraction of the levator ani (elevator muscle of the anus), which is often involved in the spasm.

DILAGENT is used in combination with particular pelvic muscle rehabilitation exercises (Kegel exercises) that replace the vaginal weights or cones. DILAGENT is also indicated in the treatment of post-surgical dyspareunia.

The transfer of heat to said muscles achieves significant pain reduction. The best results are obtained when DILAGENT is combined with pelvic floor rehabilitation exercises (Kegel) and, in the most severe cases of vaginismus, with psychological support.

Product Code	Description	QTY
A.1004	Sapimed Dilagent Anal Dilator	Each